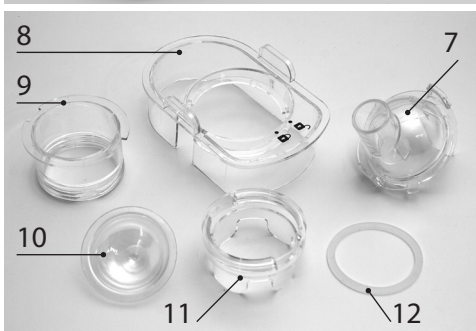
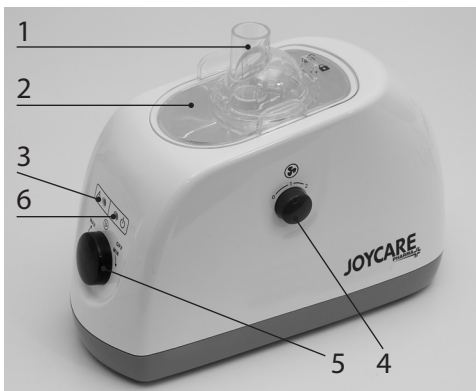
**JOYCARE<sup>®</sup>**  
PHARMA

JC-615 (NB-152U)

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### AD ULTRASUONI

#### Componenti

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## **ULTRASONIC NEBULIZER**

### **Components**

1. Nebulizing compartment
2. Water reservoir cover
3. Water level alarm light
4. Air flow adjusting knob
5. O/I and Nebulizing Rate Adjusting knob
6. Power indicator
7. Cover of nebulizing compartment
8. Water reservoir cover
9. Middle shell of nebulizing compartment
10. Medicine cup
11. Bottom shell of nebulizing compartment
12. Seal ring

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**ACCESSORIES:**

1. Adult mask
2. Child mask
3. Mouthpiece
4. Nosepiece
5. Tube

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Uchovejte v suchu

**HOUNSUN (NATONG) Co., Ltd.**

Address: No. 8 Tongxing Road, Economic & Tecnological Development Area 226009 Nantong City , PEOPLE'S REPUBLIC OF CHINA

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### Guidance of Manufacturer's Declaration-Electromagnetic Emissions

The model JC-615(NB-152U) is intended for use in the electromagnetic environment specified below. The customer or the user of the model JC-615 (NB-152U) should assure that it is used in such an environment

EMISSION TEST	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT- GUIDANCE
RF emissions CISPR 11	Group 1	The model JC-615(NB-152U) uses RF energy only for its internal function. therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The model JC-615(NB-152U) is suitable for use in all establishments including domestic establishments its and those directly connected to the public low power supply network that supplies buildings used for domestic purposes
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuation/ flicker emissions IEC 61000-3-3	Compliance	

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT- GUIDANCE
Electrostatic discharge IEC 61000-4-2	$\pm 6\text{kV}$ contact $\pm 8\text{kV}$ air	$\pm 6\text{kV}$ contact $\pm 8\text{kV}$ air	Floor should be wood, concrete or ceramic tile. If floors are covered with synthetic material ,the relative humidity should be 30%
Electrical fast transient / burst IEC 61000-4-4	$\pm 2\text{kV}$ for power supply lines	$\pm 2\text{kV}$ for power supply lines	Mains power quality should be that of a typical commercial or hospital environment
Surge IEC 61000-4-5	$\pm 1\text{kV}$ differential mode	$\pm 1\text{kV}$ differential mode	Mains power quality should be that of a typical commercial or hospital environment

<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p>&lt; 5% Ut (&gt;95% dip in Ut) for 0.5cycle 40% Ut (&gt;60% dip in Ut) for 5 cycle 70% Ut (30% dip in Ut) for 25cycle &lt; 5% Ut (&gt;95% dip in Ut) for 5 s</p>	<p>&lt; 5% Ut (&gt;95% dip in Ut) for 0.5cycle 40% Ut (&gt;60% dip in Ut) for 5 cycle 70% Ut (30% dip in Ut) for 25 cycle &lt; 5% Ut (&gt;95% dip in Ut) for 5 s</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the model JC-615 (NB-152U) image intensifier requires continued operation during power mains interruption, it is recommended that the Model JC-615(NB-152U) image intensifier be powered from an uninterruptible power supply or a battery</p>
<p>Power frequency magnetic field IEC 61000-4-8</p>	<p>3A/ m</p>	<p>3A/ m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT- GUIDANCE
conducted RF IEC 61000-4-6	3Vrms 150kHz to 80MHz	3Vrms	Portable and mobile communication equipment should be used no closer to any part of the model JC-615(NB-152U) including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	10V/m 80MHz to2.5GHz	3V/m	<p>Recommended separation distance</p> $d = \frac{3.5}{V_i} \sqrt{P}$ $d = \frac{3.5}{E_i} \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \frac{7}{E_i} \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where P is the maximum output power rating of transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths fixed RF transmitter as determined by an electromagnetic site survey* should be less than the compliance level in each frequency range.</p> <p>*Interference may occur in the vicinity of equipment marked with following symbol:</p> 
NOTE 1: At 80MHz and 800MHz, the higher frequency applies.			
NOTE 2: these guidelines may not apply in all situations. Electromagnetic propagation is effected by absorption and reflection from structures, objects and people.			
* field strength from fixed transmitters such as base stations for radio (cellar/cordless) telephones ,land mobile radio, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To access to electromagnetic environment duo to fixed RF transmitter, an electromagnetic site survey should be considered. If the measured field strength in the location in which the model JC-615(NB-152U) is used exceeds the applicable RF compliance level above, the model JC-615(NB-152U) should be observed, to very normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Model			
* Over the frequency range 150 kHz to 80MHz field strengths should be less than 3 V/m			

**Recommended separation distances between portable and mobile RF communications equipment and the JC-615(NB-152U)**

The model JC-615(NB-152U) is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the model JC-615(NB-152U) can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the model JC-615(NB-152U) as recommended below, according to the maximum output power of the communications.

RATED MAXIMUM OUTPUT POWER OF TRANSMITTER W	SEPARATION DISTANCE ACCORDING TO FREQUENCY OF TRANSMITTER M		
	150KHZ TO 80MHZ $d = \frac{3.5}{V_i} \sqrt{P}$	80MHZ TO 800MHZ $d = \frac{3.5}{E_i} \sqrt{P}$	800MHZ TO 2.5GHZ $d = \frac{7}{E_i} \sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.689	3.689	7.379
100	11.667	11.667	23.333

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters(m) can be estimated using the equation applicable to the frequency of transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1 At 80MHz and 800MHz, the separation distance for higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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