# **REF** EXCLUSIVE-TIM LIGHT

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***BRUGSANVISNING***

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Product code: EXCLUSIVE-TIM LIGHT

### 1. Characteristics

The wheelchair is made of a lightweight aluminum frame. It features height-adjustable profiled armrests and an optional detachable upholstery (for backrest and seat) made of a breathable mesh material. The wheelchair is equipped with a quick release function, centre of gravity adjustment on front and back wheels, attendant brakes and a foldable backrest. It has solid rubber 24" rear wheels and 8" front wheels.

**The set includes:** wheelchair frame with wheels, seat and backrest, user belt, assembly keys.

**Optional accessories:** anti-tip wheels, footrests with safety tape, mesh up upholstery.

#### Elements of the wheelchair:

- A. Hand grip
- B. Attendant brake
- C. Backrest lock
- D. Armrest
- E. Armrest adjustment
- F. Parking brake
- G. Handrim
- H. Quick release
- I. Footrest
- J. Anti-tip wheels
- K. Centre of gravity adjustment

### 2. Application

The wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self-propelled wheelchair can also be pushed by an accompanying person. The wheelchair is suitable for indoor and outdoor use.

### 3. Assembly

#### UNFOLDING AND ASSEMBLY

##### Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair using the armrests. Then, press the front and rear edge of the seat so that all the elements fit together.

### **Step 2. Backrest and seat**

Lift the backrest upwards and lock it with backrest locks which are located below the hand grips. Fix the upholstery on the wheelchair and fasten the lower part of the upholstery with the use of velcro under the seat.

### **Step 3. Footrests**

Fix the plastic grip of the footrest on the outer side of wheelchair frame. Then, put the assembly element of the footrest into the opening in the front of the frame turning the footrest inwards until it is locked.

### **Step 4. Armrests**

The armrests are adjustable in height with the use of a pin on the outer side of the armrest.

### **Step 5. Height adjustment of footrests**

Adjust the right and the left footrest by tightening the screw which is placed on the footrest tube with included key. Proper space should be left between the footplates and the ground as footplates set too low may cause difficulties in overcoming such obstacles as curbs. Adjust the footplates so that the angle between trunk and thighs is  $90^{\circ}$  (optimum sitting position).

### **Step 6. Assembly of protective strap**

Put the protective strap on the upper part of the footrest to prevent user's legs from sliding down the footplates.

## **FOLDING**

The wheelchair can be re-folded in order to be transported and stored more easily.

- Swing away the footplates and set them in vertical position.
- Detach the upholstery from the seat.
- Use the handle to lift the seat upwards until the frame of the wheelchair is folded.
- To fold the backrest, press backrest locks and pull the backrest downwards.

## **PARKING BRAKE**

To lock the wheels, push the parking brake lever forward until the wheel is fully blocked (separately for left and right wheel). Please, note that the wheel lock only serves to lock the wheels when the wheelchair is at a complete stop.

## **FOLDING BACKREST**

To fold the backrest, simultaneously press locking elements which are located under the hand grips at the back of the backrest and pull the backrest downwards.

## **ATTENDANT BRAKES**

The levers of the brakes are located behind the backrest, under the hand grips on the right and left side, and are connected with wheel lock by cable.

- In order to brake, squeeze the lever of the brake and in order to release the brake, drop the levers. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with a parking lock. To apply it, grasp the brake lever, squeeze it and pull out the locking element.

## **PARKING BRAKE ADJUSTMENT**

The correct lock of the parking brake depends on its adjustment and position. The handle of the wheel lock should be positioned so that the brake shoe blocks the wheel firmly and the lever does not face excessive resistance.

- In case of excessive resistance, pull the handle slightly forward by twisting off the locking screw which is located on the other side of the parking brake (from the inside of the wheelchair). Set the brake shoe in such position that it blocks the wheel easily without excessive resistance.
- When the parking brake shoe does not touch or lock the wheel sufficiently, follow the above instruction by moving the handle in the opposite direction.

## **4. BEFORE USE**

### **Step 1. Checking the condition of the wheelchair**

Check the condition of the wheel locks, construction and tires. Also, check also the correctness of the assembled components and their adjustment. **Do not** use the wheelchair if any defects or deficiencies are noticed.

### **Step 2. Engaging the wheel locks**

Engage the wheel locks when getting in or out of the wheelchair.

### **Step 3. Setting the footplates**

The footplates should be in vertical position when getting in or out the wheelchair. Note! Do not stand on the footplates due to the risk of damage. Do not use the footplates as a support while getting in or out the wheelchair due to the risk of tipping over and falling.

### **Step 4. Setting the wheels**

Front wheels should be positioned in the direction of riding.

### **Step 5. Getting in the wheelchair**

The wheelchair should be located at the place of changing the seat as close as possible. The wheelchair should touch the bed or other seat being used, rear wheel and front element of the frame. If the user cannot change his/her seat independently, transfer the user to the wheelchair.

#### **Step 6. Unfolding the footplates**

When the user is sitting in the wheelchair, unfold the footplates parallel to the ground so that his/her feet are on the footplates while using the wheelchair.

#### **Step 7. Correct position of the user**

The user should sit in the upright position and his/her knees should be bent at the angle of 90°.

**The wheelchair is ready to use.**

### **5. Technical data**

|                             |                                                      |
|-----------------------------|------------------------------------------------------|
| Seat dimensions             | 43 / 46 cm                                           |
| Overall width               | 64 cm for width 43 cm<br>67 cm for width 46 cm       |
| Overall depth               | 106 cm                                               |
| Seat height from the ground | 50 cm                                                |
| Overall height              | 91 cm                                                |
| Weight                      | 10,80 kg for width 43 cm<br>12,00 kg for width 46 cm |
| Maximum capacity            | 136 kg                                               |
| Folded width                | 34 cm                                                |
| Frame color                 | red, navy blue                                       |

### **6. Notes**

**Timago International Group does not bear responsibility of improper use of the wheelchair, failure to comply safety regulations and misuse.**

### **7. Cleaning and maintenance**

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.

- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.

## 8. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

## 9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 10. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

## 11. Labels

|                                                                                     |                                                                                                    |                                                                                     |                             |
|-------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-----------------------------|
|  | Reference number                                                                                   |  | Manufacturer                |
|  | LOT number                                                                                         |  | UDI code                    |
|  | Serial number                                                                                      |  | Medical device              |
|  | Note                                                                                               |  | Please read the instruction |
|  | The product meets the legal requirements of the European Union for medical devices (MDR 2017/745). |                                                                                     |                             |

Our complete line of products is available from our distributors. Learn more at: **[www.timago.com](http://www.timago.com)**. Thank you for choosing Timago!

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