Gebruuchsinforation und Wartungsanweisung des Andy Flow® Verneblers.
Instrucciones de uso y mantenimiento de la ampolla vaporizadora Andy Flow®

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Recommended use:
Pisper A: treatment of the upper respiratory tract
Pisper B: treatment of the tracheo-bronchial tract
Pisper C: treatment of the lower respiratory tract

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Do not use in presence of flammable substances such as anesthetic, oxygen or nitrogen protoxide substances.
Do not use in presence of flammable substances such as anesthetic, oxygen or nitrogen protoxide substances. Not suitable for anesthesia or pulmonary ventilation.
After every use wash the atomizer carefully following the instructions described in paragraph "Cleaning and Conservation".
Keep out of reach of children under age 6 as some parts could be swallowed.
The atomizer is a single-patient device only.

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The atomizer must be used with a standard norm air compressor.
The values of suggested flow and pressure are indicated in Table 1.
Keep away from sources of heat and direct sunlight.
Before beginning treatment with the medicine make sure of the working order by simply filling the container with 2 ml of physiological solution and spraying for several seconds.

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The Andy Flow atomizer is classified as a class IIa medical device and is certified CE0051 respecting European norms on medical apparatus 93/42 and the EN 13544-1 standard.

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In the following description refer to the numbers of the elements as indicated in fig. 1.

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
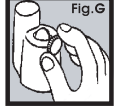
Fig. 7
After use unscrew the atomizer (as in fig. 1) and clean all the parts in warm neutral soapy water; rinse carefully (see fig. H). Remove excess water with a soft cloth and let dry in a clean area. For eventual sterilization place the atomizer in a cold sterilizing solution following the sterilizing solution's instructions. For each patient the atomizer can be used up to 6 months or 120 treatments maximum. Do not clean in boiling water or use vapor to sterilize the atomizer.

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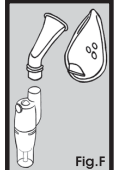Pulizia e conservazione.
Cleaning and conservation
Nettoyage et conservation.

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Apply the chosen treatment accessory: breathing mask (for adults or pediatric) or mouth piece (as in fig. F).
Regulate the valve (as in fig. G) keeping in mind the characteristics described in tables 2 and 3.

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ON = open valve
OFF = closed valve
The output test was made with 2.5 ml of NaCl solution of 1% (in conformity with the attached CC of the EN13544-1 norm) and with compressed air flow of minimum 5 l/min. and maximum at 8 l/min.

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
Analisi granulometrica
Granulometric analysis
Analyse de granulométrie

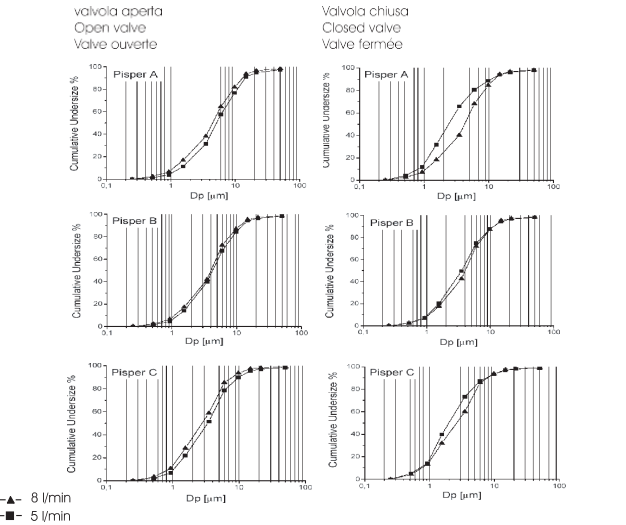5	A	ON	5.1	2.6	
		OFF	2.5	2.8	
	B	ON	4.2	2.5	
		OFF	3.8	2.5	
	C	ON	3.4	2.5	
		OFF	2.0	2.4	
8	A	ON	4.4	2.8	
		OFF	4.3	2.7	
	B	ON	4.0	2.4	
		OFF	4.0	2.5	
	C	ON	2.7	2.3	
		OFF	2.6	2.4	

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Med 2000 S.r.l.